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## Instructions for Use

### RIELLEN'S PROSTHETIC COMPONENTS FOR DENTAL IMPLANT

English

IFUI Rev00 of 02/2017

#### Instruments

Riellens Instruments are used specifically by qualified dentists, with specific training in implant dentistry, including diagnosis, pre-surgery plan Ning and surgical protocol.

#### Intended Use

The Riellens Instruments serve to screw and unscrew screws, healings, protective cover and implants cover.

#### Indication for Use

Components	Description	Models	Indication for Use
	Slotted Digital Key	Short, Medium and Long	Used to tighten the screws of prosthetic components when their fits are slotted
	Digital Hexagonal Key 0.9	Short, Medium and Long	Used to tighten the screws of prosthetic components when their fits are hexagonal
	Digital Hexagonal Key 1.2	Short, Medium and Long	
	Digital Square Key	Short, Medium and Long	Used to tighten the screws of prosthetic components when their fits are square.
	Digital Unigrip Key	Short, Medium and Long	Used to tighten the screws of prosthetic components when their fits are unigrip-shaped
	Digital Key for Mini Conical Abutment	Hexagon 2.0mm	Used to tighten the screws of Mini Conical Abutment, Conical Abutment and Immediate Provisional Abutment.
	Digital Key for Mini Conical Abutment	Hexagon 2.5mm	
	Hexagon Torque Key 0.9	Short, Medium and Long	Manufactured for use with torque Keyes (instruments for torque application and verification), these are used to apply the set torque values to the screws of prosthetic components
	Hexagon Torque Key 1.2	Short, Medium and Long	
	Square Square Key	Short, Medium and Long	
	Unigrip Square Key	Short, Medium and Long	
	Hexagon Torque Key for Counter-Angle 0.9	Short, Medium and Long	Used for the placement and pre-torque of screws and prosthetic components.
	Hexagon Torque Key for Counter-Angle 1.2	Short, Medium and Long	
	Square Torque Key for Counter-Angle	Short, Medium and Long	
	Unigrip Torque Key for Counter-Angle	Short, Medium and Long	

#### Specifications and technical features

Riellens Instruments are manufactured in Stainless Steel ASTM F899-09 (Standard Specification for Wrought Stainless Steels for Surgical Instruments) ISO 5832-1 (Standard Specification for Implants for Surgery – Metallic materials Wrought Stainless Steels), have cylindrical format, are reusable and provided non-sterile.

#### Packaging Packing

Riellens Instruments are packed individually in blister packs, ensuring the integrity and cleaning of the product until the use by the professional. Information such as: description and fitting are available in the product's label.

Riellens Instruments may not be used if the package (blister) is damaged.

**CAUTION** - Always use the package grip point, avoiding falls and direct contact with contaminated sites.

#### Contraindications:

This product does not present contraindications, as long as used correctly for the purposes indicated.

#### Handling

The choice of the Keys must be based on the following considerations: screws, implant cover, protective cover and healings: fit the digital wrench in the component fit that can be hexagonal, square, unigrip or slotted. Press the wrench and move clockwise to insert and counterclockwise to remove. For greater security, pass a dental floss through the hole of the ring on the digital key avoiding the release of the product in the mouth, making sure that the pieces are not swallowed or aspirated by the patient.

#### Cleaning of instruments:

Riellens Instruments are provided clean. Any used component that must be returned to the after-sales service must be sent after cleaning and decontamination in the dental clinic.

The Instruments must be cleaned, disinfected and sterilized before each use.

- Put the pieces in the cleaning bath for removal of organic matter from the instruments as soon as possible after use; Use enzymatic detergent, following the instructions for dilution and exposure time determined by the manufacturer.
- Perform the mechanical removal of the film of the instruments by manual brushing (soft brushes) or vibrations produced by ultrasonic cleaner.
- Wash with clean water to remove chemical waste and detergents.
- Dry the instruments always using disposable soft lint-free cloth.

Note - The responsible must be equipped with individual protection equipment (gloves, masks, goggles, aprons, hats, etc.).

#### Sterilization

These products are not sterile, thus they should be autoclaved as professional protocol and manufacturer's instructions before reuse on the patient.

Riellens recommends placing the components in the proper package for sterilization in autoclave. According to the following table:

Method	Conditions
Autoclave, Humid heat	134° for 18min

#### Precautions and Warnings

The use of incompatible components and instruments voids all warranties. Users are responsible for the proper use and handling of this product. Riellens will not be responsible for any damage caused due to inadequate use of its products.

The prosthesis installation environment must be clean and appropriate to the handling and use of health products.

**WARNING**  
Dental product intended for use by licensed professionals. Provided NON-STERILE.  
Must be sterilized before each use.

#### Warnings

Sterilize the product the day before or on the day of the procedure, the sterilization validity is from 7 to 15 days, provided the product is packaged in dry, clean environment and away from sunlight.

#### Caution

To torque the screw use the torque key to obtain the desired torque. Follow the equipment manufacturer's instructions.

#### Adverse Effects

Adverse effects will occur only if the choice of the instruments by professionals is inadequate.

#### Storage Conditions

Always store it at room temperature between 15°C and 35°C in the original package. Do not expose to direct sunlight.

#### Product disposal

The legal procedures in force must be complied with for disposal of Riellens Instruments deemed inadequate and they must also have their characteristics removed by being cut, twisted or filed.

#### Expiration Date and batch

Manufacture date, expiration and batch see package. This product shall be disposed if it loses its functionality.

#### WARNING INSTRUCTIONS FOR USE

These INSTRUCTIONS FOR USE are available online at the manufacturer's website [www.riellens.com.br](http://www.riellens.com.br)

The INSTRUCTIONS FOR USE are indexed on the website by the TRADE NAME of the product, which can be found on the label of the product purchased.

The INSTRUCTIONS FOR USE can be provided in printed material without additional cost if the professional prefers. Such request can be made at the Customer Service of the manufacturer.

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