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# Instruction for Use RIELLEN'S PROSTHETIC COMPONENTS FOR DENTAL IMPLANT

English

IFUI Rev01 of 10/2020

## Instruments

Riellens Instruments are for the specific use of qualified dentists, with specific training in implantology, including diagnosis, preoperative planning and surgical protocol.

### 1. Intentions of Use

Riellens Instruments have the function of assisting in the installation or removal of components and implants.

### 2. INDICATION FOR USE

Table I - Instruments

Wrenches	INDICATION FOR USE	Model	Description	Size
	Used for capturing, transporting, fixing and removing screws and prosthetic components.	Screw driver	Digital Screwdriver Screw driver Manual Hex 0.9 Screw Driver Square Screw Driver Unigrip Screw Driver Torx	Short, Medium and Long
		Mini Abutment Screw driver	Screw Driver Mini Conical Abutment 2.0mm Screw Driver Mini Conical Abutment 2.5mm	-
		Torque Driver	Torque Driver Hex 0.9 Torque Driver Hex 1.2 Torque Driver Square Torque Driver Unigrip Torque Driver Torx Square Adapter Torque Driver	Short, Medium and Long
		Mini Abutment Torque Driver	Torque Driver Mini Conical Abutment 2.0mm Torque Driver Mini Conical Abutment 2.5mm	-
		Screw Driver Machine	Screw Driver Machine Hex 0.9 Screw Driver Machine Hex 1.2 Screw Driver Machine Square Screw Driver Machine Unigrip Dynamic Driver Machine	Short, Medium and Long
	Used for capturing, transporting, extracting and fixing the implant	Implant Driver	Implant Driver 3.0, NP and RP NA Implant Driver NP, RP and WP HE B Implant Driver NP, RP and WP AT Implant Driver 3.0, NC and RC BL Implant Driver MS Implant Driver CMN	Short, Medium and Long
		Zygomatic	Zygomatic installation Drive 45° HE Zygomatic installation Drive 0° HE	-
		Extractor	Implant Extractor NP/RP	-

### 3. Technical specifications and characteristics

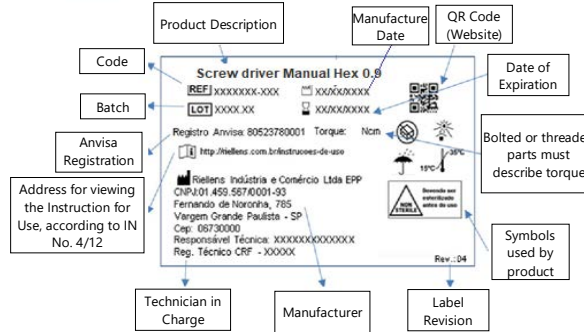
Riellens Instruments are made of Stainless Steel ASTM F899 (Standard Specification for Wrought Stainless Steels for Surgical Instruments) ISO 5832-1 (Standard Specification for Implants for Surgery – Metallic materials Wrought Stainless Steels), they have a cylindrical shape, are reusable and provided non-sterile.

### 4. Way of Presentation

#### Packaging

Riellens Instruments are individually packaged in blister packaging, ensuring the integrity and cleanliness of the product until the use by the professional. Information such as description and fitting are available on the product label.

**CAUTION** - Always use the handle spot of the package, avoiding falls and direct contact with contaminated places.

### 5. Handling

To use the wrench, fit the tip of the wrench into the hexagonal, square or unigrip internal socket of the screw or prosthetic component and apply the recommended torque for each component or implant. For greater safety, floss through the hole in the ring on the digital driver, or make sure that the key is well fixed to the contra-angle handpiece, prevent the release of the product into the mouth, ensuring that the parts are not swallowed or aspirated by the patient.

### 6. Instruments Cleaning:

Instruments must be clean, sanitized and sterilized before each use.

- Place the parts in the cleaning bath to remove organic matter from the instruments as soon as possible, after its use;
  - Use enzymatic detergent, following the dilution instructions and exposure time determined by the manufacturer.
  - Always use distilled or deionized water for this procedure. Heat the water between 30 and 40 °C, to facilitate cleaning.
  - Perform mechanical removal of dirt from instruments through manual brushing (brushed with soft or Nylon bristles for cleaning fittings)
  - Rinse under running drinking water to remove residues and detergents.
  - Dry the instruments always using clean soft fabric or use lint-free disposable fabric.
  - Pack in specific packaging for sterilization in an autoclave.
- Note - The person responsible must be gowned with protective personal equipment (gloves, masks, goggles, aprons, hats, etc.).

### 7. Sterilization

These products are provided non-sterile; thus, they shall be sterilized in an autoclave.

- Use chemical and biological indicators for each sterilization cycle, according to practice protocol.
- Allow Riellens Instruments to dry and cool in the autoclave before handling to avoid contamination and oxidation of materials.
- Riellens recommends sterilization at the parameters of 134 °C for 15 minutes.

### 8. Contraindications

This product has no contraindications, as they are used correctly for the indicated purposes.

### 9. Adverse Effects

Adverse effects will only occur if the choice of instruments by the professionals is inadequate.

### 10. Precautions and Warnings

- The use of components or instruments that are not compatible will render all warranties void. The correct use and handling of this product is the sole responsibility of the user. Riellens takes no responsibility for damages resulting from incorrect use of its products.
- The environment for installing the prosthesis must be clean and suitable for the handling

- The product must be used only by qualified dental professionals who already have all scientific information necessary for the correct use of the product.

### ATTENTION

Dental product intended for use by qualified professionals.  
Provided NON-STERILE, never use Riellens instruments without performing the cleaning and sterilization process after each use, risk of infection with infectious agents.  
Caution when handling the instruments, risk of accidentally aspirating or swallowing.

### 11. Alerts / Cautions

- Never use saline solution, especially Sodium hypochlorite and physiological saline for cleaning or rinsing the surgical instruments, to avoid the corrosion process.
- Never use steel wool or sponges and abrasive products, so that the instruments are not damaged.
- Avoid friction between instruments when cleaning to avoid deformation of smaller and delicate parts.
- Never allow the instrument to dry naturally, the humidity favors corrosion of the material.
- Never overlap the instruments in the autoclave, it may affect the sterilization effectiveness.

### 12. Useful Life

Do not use the instruments if you notice cracks, wear or oxidation/corrosion points. This may lead to problems in the performance of the instruments. This product is recommended for up to 30 uses, provided that the conditions of use recommended by Riellens are observed. Regardless of the number of times the instruments has been used, the professional must always evaluate their conditions after each use.

### 13. Storage Conditions

Always store at room temperature between 15 °C and 35 °C in the original package. Do not expose to direct sunlight.

### 14. Product disposal

For disposal of Riellens Instruments deemed unsuitable for use, they must follow the legal procedures in force for product disposal and must be de-characterized by being cut, bent, or filed.

### 15. Expiration Date and Batch

See the manufacture date, expiration and batch on the package. This product shall be disposed if it loses its functionality.

### 16. INSTRUCTION FOR USE ALERT

This INSTRUCTION FOR USE is available in non-printed (electronic) format, through the electronic address of the manufacturer at [www.riellens.com.br](http://www.riellens.com.br)  
The INSTRUCTIONS FOR USE are indexed on the website through the respective product TRADE NAME, informed on the label of the product purchased.  
If it is in the professional's interest, the INSTRUCTIONS FOR USE may be provided in printed form, at no additional cost. The request must be made to the SAC (Customer Service) of the manufacturer.

### 17. Symbols

	Fabricante / Manufacturing / Fabricante / Fecha de fabricación		Mantenha afastado da luz solar / Keep away from sunlight / Mantener fuera de la luz solar
	Data de fabricação / Date of manufacture / Fecha de fabricación		Não utilize se a embalagem estiver danificada / Do not use if package damaged / No utilizar si el envase está dañado
	Referência de catálogo / Catalogue number / Número de catálogo		Limite Superior de temperatura / Upper limit of temperature / Limite superior de temperatura
	Código de lote / Batch code / Código del lote		Não estéil / Non-Sterile / no estéril
	Prazo de validade / Use by / Fecha de caducidad		Conserva seco / Keep dry / Mantener seco
	Cuidado / Caution / Cuidado		Consulte as instruções de uso / Consult instructions for use / Consulte las instrucciones de uso

**RIELLEN'S**  
RIELLEN'S INDÚSTRIA E COMÉRCIO LTDA EPP.  
Rua Fernando de Noronha, 785 – Jardim Margarida CEP: 06730-000  
Vargem Grande Paulista/SP – Brazil CNPJ. 01. 459. 567/ 0001-93  
AUTHORIZ/MS: 8.05237.8  
Technical Pharmacist: Vanessa Guimarães Souza de Castro CRF: 63664  
Customer Service Phone: 55 11 - 4158-5018

Anvisa Registration: Instruments No. 80523780001 / Prosthetic kit No. 80523790001

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