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Registro Anvisa: Transfer N° 80523780002 e N° 80523780003

Instruction for Use RIELLEN'S PROSTHETIC COMPONENTS FOR DENTAL IMPLANT

English

IFUT Rev02 10/2020

Transfer

Riellens Transfers are for the specific use of qualified professionals, with specific training in implantology, including diagnosis, preoperative planning and surgical protocol.

1. Intentions of Use

Riellens Transfers have the function of transferring (impression) on Conical Abutments, Mini Conical Abutment and Implants. It is indicated for open tray (drag) or closed tray (replacement) techniques.

2. INDICATION FOR USE

| RIELLEN'S TRANSFER COMPONENTS | | | | | |
|---|--|-------------|--|----------------|--|
| COMPONENT | INDICATION FOR USE | MODELS | CHARACTERISTICS | CONNECTI ON | DIAMETER |
|  Stainless Steel Transfer | Used to transfer the positioning of the dental implant installed in the oral cavity to the plaster model, by means of an impression. | Open Tray | Used for impressions of single and multiple prosthetics with open tray | HE HI TI | NP, RP, WP Ø3.3, 3.75 Ø 3.3 |
| | | Closed Tray | Used for impressions of single and multiple prosthetics with Closed tray | CM | Ø3.0, Ø 3.8, Ø4.5, Ø4.5/5.0, RN, WN |
|  Titanium Transfer | | Open Tray | Used for impressions of single and multiple prosthetics with open tray | HI TI | NP,RP,WP Ø 3.5/4.0, 4.5/5.0, NP,RP,WP |
| | | Closed Tray | Used for impressions of single and multiple prosthetics with Closed tray | CM | Ø 2.9, 3.8, NN, NP, RN, RP, WN, NC, RC |


3. Technical specifications and characteristics

Riellens Transfers are made of titanium according to standards ASTM F136 and ISO 5832-3 (Standard Specification for Wrought Titanium-6Aluminum-4Vanadium) or Stainless Steel ASTM F899-09 303 (Standard Specification for Wrought Stainless Steels for Surgical Instruments) ISO 5832-1 (Standard Specification for Implants for Surgery – Metallic materials Wrought Stainless Steels), intended for single use, its reuse is forbidden, provided non-sterile and have a cylindrical in shape.

They are presented in two parts, Transfer + Screw and are available as open tray and closed tray in relation to the impression technique.

4. Way of Presentation Packaging

Riellens Transfers are individually packaged in blister packaging, ensuring the integrity and cleanliness of the product until the use by the professional. Information such as description, dimension, date of manufacture and batch are available on the product label. Information such as description, dimension and code are available on the product label.

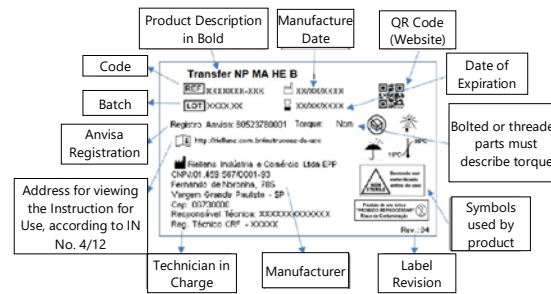In the event of damage to the package (Blister) or after the expiration date, Riellens Transfer cannot be used. All Riellens Transfer are sold with their respective bolt, which can be sold separately as a spare part.

CAUTION - Always use the handle spot of the package, avoiding falls and direct contact with contaminated places.

5. Handling

Place the selected Transfer according to the planned impression technique and install directly on the implant or intermediate. To tighten the screw, use manual torque with digital wrench. Use the impression technique to obtain a mold with the implant exact position. Remove the Impression of the patient's mouth and pour the plaster to obtain replica of the implant in the oral cavity.

6. Contraindications

This product has no contraindications, as they are used correctly for the indicated purposes.

7. Component cleaning

Riellens makes available all of its Transfers sanitized.

8. Sterilization

These products are provided non-sterile; thus, they shall be sterilized in an autoclave. - Use chemical and biological indicators for each sterilization cycle, according to practice protocol. - Allow Riellens Transfer to dry and cool in the autoclave before handling to avoid contamination. - Riellens recommends sterilization at the parameters of 134 °C for 15 minutes. - Riellens recommends placing the components in their own packaging for sterilization in an autoclave.

9. Precautions and Warnings

The use of components or instruments that are not compatible will render all warranties void. The correct use and handling of this product is the sole responsibility of the user. Riellens takes no responsibility for damages resulting from incorrect use of its products. The environment for installing the prosthesis must be clean and suitable for the handling and use of health products.

ATTENTION
Dental product intended for use by qualified professionals.
Supplied NON-STERILE and must be sterilized before use; risk of infection with infectious agents.

10. Caution

To tighten the screw, use a manual digital wrench until the desired torque is obtained. Ensure that the transfers screws are correctly aligned between the implant and/or intermediate to avoid thread wear.

11. Adverse Effects

This product has no adverse effects, if used correctly for the indicated purposes.

12. Storage Conditions

Always store at room temperature between 15 °C and 35 °C in the original package.

13. Product disposal

For the disposal of Transfers used or considered inadequate, it is necessary to follow the current legal procedures for the disposal of products, and they must be defused by being cut, bent or filed away.


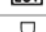
14. Expiration Date and Batch

See the manufacture date, expiration and batch on the package. This product shall be disposed if it loses its functionality.

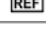15. INSTRUCTION FOR USE ALERT

This INSTRUCTION FOR USE is available in non-printed (electronic) format, through the electronic address of the manufacturer at www.riellens.com.br. The INSTRUCTIONS FOR USE are indexed on the website through the respective product TRADE NAME, informed on the label of the product purchased. If it is in the professional's interest, the INSTRUCTIONS FOR USE may be provided in printed form, at no additional cost. The request must be made to the SAC (Customer Service) of the manufacturer.

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Anvisa Registration: Transfer No. 80523780002 and No. 80523780003

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