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## Instructions for Use

### RIELLENS PROSTHETIC COMPONENTS FOR DENTAL IMPLANT

English

FIUUC Rev01 of 10/2019

#### Ucla / Copings

Riellens Ucla and Copings are of specific use for qualified professionals, with specific training in dental prostheses and dental implant, including diagnosis, preoperative planning and surgical protocol.

#### 1. Purposes of Use

Riellens Ucla and Copings have the purpose of replacing or substituting the tooth structure in order to restore the patient's chewing and aesthetic functions.

#### 2. Table 1 - Indication for Use

RIELLEN'S GLOVE / COPING COMPONENTS						
COMPONENT S	INDICATION FOR USE	MODELS	CHARACTERISTICS	CONN ECTI ON	DIAMETER	
	Used for waxing and casting single or multiple screwed prosthetic crown structures.	CrCo	<p>The Cobalt-chrome base does not change in its structure, offering greater precision in adapting the prosthesis on the implant.</p>	Anti-Rotational	Used for single prosthetic crowns	HE NP, RP, WP
			Rotational	Used for multiple single prosthetic crowns	HI Ø 3.3, 3.75, 3.5/4.0, 4.5/5.0, NP, RP, WP	
		Plastic	Product undergoes degradation in the casting process	Anti-Rotational	Used for single prosthetic crowns	TI Ø 3.3, 4.0, NP, RP, WP
				Rotational	Used for multiple single prosthetic crowns	CM Ø 3.0, 3.8, 4.5, 4.5/5.0, NN, NP, RN, RP, WN, NC, RC

**KEY:** HE: External Hexagon; HI: Internal Hexagon; CM: Morse Taper; TI: Internal Triangle

**WARNING:** All Riellens Ucla and Copings have fittings that are dimensionally compatible with HE, HI, CM and TI connection implants.

#### 3. Specifications and technical characteristics

Ucla and Copings are available entirely in plastic, manufactured in Natural Polyacetal and with a CrCo strap, manufactured based on Cobalt-chrome in compliance with the Standard Specification for Wrought Cobalt-28Chromium-6Molybdenum Alloys for Surgical Implants (UNS R31537, UNS R31538, and UNS R31539) and have a plastic glove in a cylindrical shape attached to the top section of the base, they are for single use and non-sterile, presented in two parts, component + screw for fixation.

#### 4. Method of Presentation

##### Package

Riellens Ucla and Copings are individually packaged in blister packaging, ensuring the integrity and cleanliness of the product until the use by the professional. Information such as description, dimension is available on the product label. They are accompanied by three traceability tags, which must be fixed: (i) in the clinical record; (ii) in the document to be delivered to the patient; (iii) in the product purchase invoice.

In the event of damage to the package (blister) or after the expiration date, Riellens Ucla and Copings cannot be used. All Riellens Ucla and Copings are sold with their respective screw, which can be sold separately as spare part.

**CAUTION** - Always use the handle spot of the package, avoiding falls and direct contact with contaminated places.

#### 5. Handling

- Prosthetic

After the clinical sequence and the preparation of the work model, start the prosthetic planning with waxing, defining the final anatomy of the Ucla and Copings. The edges determined by the machining process must be observed. Waxing beyond the edges of the components is not recommended, so that there is no danger of invasion of the area by the metal during the casting process. For fusion-bonded coatings, follow the manufacturer's guidelines.

- Dentist

Prosthesis directly on the implant or on the intermediate - After finishing the casting process, the prosthesis must be cleaned according to the professional protocol. Place the prosthesis directly on the implant or intermediate and then insert the screw, applying the recommended torque. (specific for each prosthetic interface), according to the recommended torque table.

#### 6. Recommended Torques / Used Wrenches

**Table 2 - Correspondence Table of Screws Torques for Glove and Coping components.**

COMPONENTS	TYPES OF THREADS	PLATFORM		RECOMMENDED TORQUE	MAXIMUM ALLOWABLE TORQUE	USED WRENCH		
		HE	MU(RP) NA(Ø3.0) ES (RP)			HEX. 1.2	UNIG (Torx)	SQUARE 1.3
	Thread M1.4	HE	MU(RP) NA(Ø3.0) ES (RP)	20 N.cm	26 N.cm	X	X	X
		HI	HEB (NP)	25 N.cm	32.5 N.cm	X	X	X
	Thread M1.6	HI	HI CE (NP, RP, WP, Ø6.0)					
		CM	BL (NC, RC); NA (NP)					
	Thread M1.8	HE	MU(WP)	25 N.cm	32.5 N.cm	X	X	X
		HI	MS (Ø 3.75)					
	Thread M2.0	TI	RS (NP)	32 N.cm	41.6 N.cm	X	X	X
		CM	CMN (Ø 3.3, 4.5) CMD (Ø 3.8)					
	Thread M2.5	HE	3HE (NP; WP) HEB(RP)	32 N.cm	41.6 N.cm	X	X	X
		TI	RS (RP, WP, Ø 6.0)					
	Thread M2.5	CM	TL, ST, SI (RN, WN) NA (RP) CMD (Ø 4.5/ 5.0)	32 N.cm	41.6 N.cm	X	X	X
		TI	RS (RP, WP)					

#### 7. Contraindications

The use of Riellens Ucla and Copings is contraindicated for patients who are sensitive to the material, with the possibility of allergic reactions to any of the chemical elements that comprise their raw material, such as: Al, Co, Cr, Mo, Ni, Mg, Fe, Si.

#### 8. Cleaning of the Components

Riellens provides all its Ucla and Copings sanitized. Any component used that must be returned to the after-sales service shall be shipped after cleaning and decontamination in the office.

#### 9. Precautions and Warnings

Failure to follow the procedures outlined in this instruction for use may lead to one or all of the following complications: implant or component damage; components loosening; inadequate final restoration or poor adaptation of mobile and fixed prostheses; implant failure. Riellens RECOMMENDS SINGLE USE. REPROCESSING IS FORBIDDEN. The reuse of the components will result in damage to the patient's health, who may come in contact with a contaminated product. The use of components or instruments that are not compatible will render all warranties void. The correct use and handling of this product is the sole responsibility of the user. Riellens takes no responsibility for damages resulting from the incorrect use of its products. The environment for the implementation of the prosthesis must be clean and suitable for the handling and of health products.

**ATTENTION**  
Dental product intended for use by qualified professionals.  
Supplied NON STERILE. Casting components must not be sterilized in an autoclave as they do not support such temperatures.

#### 10. Warnings

Torsion superior to the recommended torque may result in damage to the screw fitting and fracture of the screw.

Torsion inferior to the recommended torque may result in component loosening, which may lead to prosthesis failure.

#### 11. Caution

Use the torque meter until the desired torque is obtained to tighten the screw and use the fixing wrench to stabilize. Follow the instructions from the manufacturer of the equipment.

Ensure that the connection screws between the implant and the prosthesis are correctly aligned on all secondary components to prevent wear on the threads.

#### 12. Adverse Effects

No adverse effects will occur, provided it is used only according to its indications for use.

#### 13. Pos-implementation Care

Advise the patient to see the dental surgeon periodically.

#### 14. Condition of Storage

Always store at room temperature between 15°C and 35°C in the original package. Do not expose to direct sunlight.

#### 15. Disposal of the Product

For disposal of Riellens Ucla and Copings used or considered inadequate, it is necessary to follow the current legal procedures for disposal of products, and they must be decharacterized, whether by being cut, kneaded or filed.

#### 16. Expiration Date and Batch

See the manufacture date, expiration and batch on the package.

#### 17. INSTRUCTION FOR USE WARNING

This INSTRUCTION FOR USE is available in non-printed (electronic) format, through the electronic address of the manufacturer [www.riellens.com.br](http://www.riellens.com.br). The INSTRUCTIONS FOR USE are indexed on the website through the respective product TRADE NAME, informed on the label of the product purchased. If it is in the professional's interest, the INSTRUCTIONS FOR USE may be provided in printed form at no additional cost. The request must be made to the manufacturer's Customer Service (SAC).

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