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### Instructions for Use

### RIELLENS PROSTHETIC COMPONENTS FOR DENTAL IMPLANT

English

IFULC Rev00 of 02/2017

### Uclas / Copings

Riellens Uclas and Copings are used specifically by qualified professionals, with specific training in dental prosthesis and implant dentistry, including diagnosis, pre-surgery planning and surgical protocol.

### Intended Use

Riellens Uclas and Copings have as objective to replace the dental structure in order to restore the chewing and aesthetics functions of the patient.

RIELLEN'S UCLA / COPING COMPONENTS					
COMPONENTS	INDICATION FOR USE	MODELS	FEATURES	CONNECTION	DIAMETER
	Used for waxing and casting of unitary or multiple prosthetic crowns.	CrCo Base	Cobalt-Chrome base does not change its structure, offering greater precision in adjustment of the prosthesis on the implant.	Anti-Rotational	Ø 3.3, 4.1, 5.0
				Rotational	Ø 3.4, 4.1, 5.0
	Product suffers degradation in the casting process	Plastic		Anti-Rotational	Ø 3.5, 4.3, 5.0
				Rotational	Ø 3.0, Ø 3.3, 4.0, NN, NP, RN, RP, WN

### Specifications and technical features

The Uclas and Copings are available fully plastic, made of Natural Polyacetal and with strap in CrCo, manufactured based on Cobalt Chrome as Standard Specification for Wrought Cobalt-28Chromium-6Molybdenum Alloys for Surgical Implants (UNS R31537, UNS R31538, and UNS R31539) and have a cylindrical shaped plastic glove that is fitted at the top of the base, for single-use and non-sterile, presented in two parts, component + screw for fixing.

## Recommended Torques:

PROTEIN COMPONENTS	TYPES OF THREADS	INDICATION OF USE	MODELS	RECOMMENDED TORQUE	MAXIMUM TORQUE PERMITTED
	Thread M1.4	They are indicated for fixation of the components directly to the implant, with the application of controlled torque guaranteeing the perfect adaptation. They are made with ASTM F136 Titanium in a single body having at one end an external thread and in the other an internal socket in the shape of Hexagon, Square or Unigrip.	Hexagonal Square Unigrip	20 N.cm	26 N.cm
	Thread M1.6			25 N.cm	32.5 N.cm
	Thread M1.8, M2.0 e M2.5			32 N.cm	41.6 N.cm

## Packaging Packing

Riellens Ucla and Copings are packed individually in blister packs, ensuring the integrity and cleaning of the product until the use by the professional. Information such as: description, size are available in the product's label. They are accompanied by three traceability labels, which must be mandatorily fixed in: (i) the clinical record; (ii) the document to be delivered to the patient; (iii) the product's purchase invoice.

**CAUTION** - Always use the package grip point, avoiding falls and direct contact with contaminated sites.

Riellens Ucla and Copings may not be used if the package (Blister) is damaged or after the expiration date. All Riellens Ucla and Copings are marketed with their respective screw, and may be sold separately as spare part.

## Handling

### - Prosthetic

After the clinical sequence and making of the work model, start the prosthetic planning with waxing, setting the final anatomy of Ucla and Copings. The margins determined by the machining process must be respected. Waxing is not recommended beyond the margins of the components, so there is no danger of invasion of the area by the metal during the casting process. For fusion coating, follow the manufacturer's guidelines.

### - Dentist

Prosthesis directly on the implant or intermediary - After the casting process, the prosthesis should be cleaned as professional protocol. Place the prosthesis directly on the implant or intermediary and then insert the screw, applying the recommended torque (specific to each prosthetic interface), according to the recommended torque table.

## Contraindications:

The use of Riellens Ucla and Copings is contraindicated in cases of patient sensitivity to the material, with the possibility of allergic reactions to any of the chemical elements that comprise the raw material such as: Al, Co, Cr, Mo, Ni, Mg, Fe, Si.

## Components cleaning:

Riellens Uclas and Copings are provided clean. Any used component that must be returned to the after-sales service must be sent after cleaning and decontamination in the dental clinic.

## Sterilization

The casting cylinder Components shall be used in the definitive prosthesis casting and therefore the chapter "Sterilization" does not apply.

## Precautions and Warnings

The non-compliance with the procedures described in these instructions for use may cause one or all of the following complications: damage to the implant or components; components slackening; inadequate final restoration or maladaptation of fixed and movable prostheses; implant failure.

Riellens RECOMMENDS SINGLE USE, BEING PROHIBITED TO REPROCESS, the reuse of components will cause damage to the patient's health that might come into contact with a contaminated product. The use of incompatible components and instruments voids all warranties. Users are responsible for the proper use and handling of this product. Riellens will not be responsible for any damage caused due to inadequate use of its products.

The prosthesis installation environment must be clean and appropriate to the handling and use of health product.

## WARNING

Dental product intended for use by licensed professionals. **NON-STERILE**. Casting Cylinder Components should not be autoclaved as it does not support such temperatures.

## Warnings

Going above the recommended torque values may result in damage to the screw fit and fracture, causing the prosthesis to fail. Torsion lower than the recommended torque may result in slackening of the component, which can lead to prosthesis failure.

## Caution

In order to tighten the screw use the torque wrench until the intended torque is attained, and, for stabilization, use the fixation key. Follow the equipment manufacturer's instructions.

Make sure that the connections screws between the implant and the prosthesis are properly

## Adverse Effects

No adverse effects will occur, provided that it is used only for the intended indications.

## Caution Post-installation

Advise the patient to periodically visit the dental surgeon.

## Storage Conditions

Always store it at room temperature between 15°C and 35°C in the original package. Do not expose to direct sunlight.

## Product disposal

The legal procedures in force must be complied with for disposal of used or inadequate Ucla and Copings and they must also have their characteristics removed by being cut, twisted or filed.

## Expiration Date and batch

Manufacture date, expiration and batch see package.

## WARNING INSTRUCTIONS FOR USE

These INSTRUCTIONS FOR USE are available online at the manufacturer's website [www.riellens.com.br](http://www.riellens.com.br)

The INSTRUCTIONS FOR USE are indexed on the website by the TRADE NAME of the product, which can be found on the label of the product purchased. The INSTRUCTIONS FOR USE can be provided in printed material without additional cost if the professional prefers. Such request can be made at the Customer Service of the manufacturer.

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