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## Instructions for Use

### RIELLENS PROSTHETIC COMPONENTS FOR DENTAL IMPLANT

English

IFUC Rev00 de 02/2017

#### Healing / Protective Cover

Riellens Healings and Protective Cover are used specifically by qualified professionals, with specific training in implant dentistry, including diagnosis, pre-surgery planning and surgical protocol.

#### Intended Use

The Healings and Protective Cover serve to expose the implant to the oral environment, preserve and increase, if necessary, the gingive inserted and adjust the thickness of soft tissue, healing the peri-implant gingival tissue.

#### Indication of Use

RIELLEN'S HEALINGS / PROTECTIVE COVER						
COMPONENTS	INDICATION FOR USE	MODELS	FEATURES	CONNECTION	DIAMETER	HEIGHT mm
					1   2   3   4   5	
	The Healing and Protective Cover are used in the second phase of the implant surgery, with the purpose of creating a cylindrical space in the gingiva, healing the gingival tissue properly.	Parallel and Divergent	It is a single body component applied directly on the implant	HE HI TI CM	Ø 33, 4,1, 5,0 Ø 34, 4,1, 5,0 Ø 35, 4,3, 5,0 Ø3,0, Ø 3,3, 4,0, NN, NP, RN, RP, WN	_ x x x x _ x x x x _ x x x x _ x x x x
	It is a single body component applied directly over the intermediate part	Universal	RP and WP		Short, Medium and Long	

#### Recommended Torques:

PROSTHETIC COMPONENTS	TYPES OF THREADS	MODELS	RECOMMENDED TORQUE	MAXIMUM TORQUE PERMITTED
	Thread M1.4	Hexagonal or Unigrip	20 N.cm	26 N.cm
			25 N.cm	32.5 N.cm
			25 N.cm	32.5 N.cm
			32 N.cm	41.6 N.cm
			32 N.cm	41.6 N.cm

## Specifications and technical features

Riellens Healings and Protective Cover are manufactured in titanium according to ASTM F136 and ISO 5832-3 standards (Standard Specification for Wrought Titanium-6Aluminum-4Vanadium), have cylindrical shape, socket shaft (1.2mm) or unigrip in the upper part and are non-sterile.

## Packaging

### Packing

Riellens Healings and Protective Cover are packed individually in blister packs, ensuring the integrity and cleaning of the product until the use by the professional. Information such as: description, size, manufacturing date and batch are available in the product's label.

**CAUTION** - Always use the package grip point, avoiding falls and direct contact with contaminated sites.

Riellens Healings and Protective Cover may not be used if the package (blister) is damaged or after the expiration date.

## Handling

### - Healings

Place the Healings and Protective Cover under the implant and apply the recommended torque (specific for each prosthetic interface). The Healing is used for a short period being recommended its removal within 30 days after installation.

### - Protective Cover

Healings Place and under the mini abutment and conical abutment and apply the recommended torque (specific for each prosthetic interface). The Protective Cover is used for a short period being recommended its removal within 30 days after installation.

## Contraindications:

The use of Riellens Healings and Protective Cover is contraindicated in cases in which the patient experiences chronic periodontal inflammation, inadequate oral hygiene, intraoral infection, insufficient bone quality, peri-implant infection, endodontic or periodontal conditions, untreated periodontitis, smoking and allergy to any of the chemical elements that comprise the raw material, such as: V, Ti, Al, O, N, Fe, H and C.

## Components cleaning:

Riellens Healings and Protective Cover are provided clean. Any used component that must be returned to the after-sales service must be sent sterile after pre-disinfection, cleaning and decontamination in the dental clinic.

## Sterilization

The Healings and Protective Cover are not sterile, thus they should be autoclaved as professional protocol and manufacturer's instructions before use on the patient.

Riellens recommends placing the components in the proper package for sterilization in autoclave. According to the following table:

Method	Conditions
Autoclave, Humid heat	134° for 18min

## Precautions and Warnings

The non-compliance with the procedures described in these instructions for use may cause one or all of the following complications: damage to the implant; components slackening; inadequate final restoration or poor healing of gingival tissue; implant failure.

"DO NOT REPROCESS" Healings and Protective Cover, the reuse of components will cause damage to the patient's health that might come into contact with a contaminated product. The use of incompatible components and instruments voids all warranties. Users are responsible for the proper use and handling of this product. Riellens will not be responsible for any damage caused due to inadequate use of its products.

The prosthesis installation environment must be clean and appropriate to the handling and use of health products.

If there are technical complaints or adverse events with Riellens products, please notify us through the SAC.

## WARNING

Dental product intended for use by licensed professionals. Provided NON-Sterile. Must be sterilized before use.

## Warnings

Going above the recommended torque values may result in damage to the Healing and Protective Cover fit and fracture, causing the prosthesis to fail.

Torsion lower than the recommended torque may result in slackening of the component, which can lead to poor healing of the gingival tissue.

## Caution

In order to tighten the Healing and Protective Cover Use the wrench and the torque wrench until the required torque is reached. Follow the equipment manufacturer's instructions. Ensure the Healings and Protective Cover are properly aligned to prevent wear on the threads.

## Adverse Effects

Possible complications:

- Pain, swelling, phonetic difficulties, gum inflammation, located or systemic infections or aesthetic problems.

## Caution Post-installation

Advise the patient to periodically visit the dental surgeon.

## Storage Conditions

Always store it at room temperature between 15°C and 35°C in the original package. Do not expose to direct sunlight.

## Product disposal

## Expiration Date and batch

Manufacture date, expiration and batch see package.

## WARNING INSTRUCTIONS FOR USE

These INSTRUCTIONS FOR USE are available online at the manufacturer's website [www.riellens.com.br](http://www.riellens.com.br)

The INSTRUCTIONS FOR USE are indexed on the website by the TRADE NAME of the product, which can be found on the label of the product purchased. The INSTRUCTIONS FOR USE can be provided in printed material without additional cost if the professional prefers. Such request can be made at the Customer Service of the manufacturer.

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