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## Instruction for Use

### RIELLEN'S PROSTHETIC COMPONENTS FOR DENTAL IMPLANT

English

IFUC Rev01 of 10/2019

#### Healing/Protective Cover

Riellens Healing and Protective Covers are of specific use by qualified professionals, with specific training in dental implant, including diagnosis, preoperative planning and the surgical protocol.

#### 1. Purposes of Use

The Healing and Protective Covers have the purpose of exposing the implant to the oral environment, preserving and increasing, if necessary, the inserted gingiva and adjusting the thickness of soft tissues, healing the peri-implant gingival tissue.

#### 2. Table 1 - Indication for Use

RIELLEN'S HEALING/PROTECTIVE COVERS COMPONENTS										
COMPONENTS	INDICATION FOR USE	CHARACTERISTICS	CONNECTION	DIAMETER	HEIGHTS mm					
					2	3	4	5	6	7
	The Healing and Protective Covers are used in the second phase of implant surgery, with the purpose of creating a cylindrical space in the gingiva, performing the appropriate healing of the gingival tissue.	It is a single-body component applied directly to the implant	HE	NP, RP, WP	x	x	x	x	x	x
			HI	NP, RP, WP, Ø 3.75, 4.1, 4.3, 5.0, 6.0	x	x	x	x	x	x
			TI	NP, RP, WP, 6.0	x	x	x	x	x	x
	It is a single-body component applied directly to the intermediate part	It is a single-body component applied directly to the intermediate part	HE	NP, RP, WP						
			HI	NP, RP, WP, Ø 3.75, 6.0						
			TI	NP, RP, WP, 6.0						
			HE	NP, RP, WP						
			HI	NP, RP, WP, Ø 3.75, 6.0						
			TI	NP, RP, WP, 6.0						
			HE	NP, RP, WP						
			HI	NP, RP, WP, Ø 3.75, 6.0						
			TI	NP, RP, WP, 6.0						
			HE	NP, RP, WP						
			HI	NP, RP, WP, Ø 3.75, 6.0						
			TI	NP, RP, WP, 6.0						

**KEY:** HE: External Hexagon; HI: Internal Hexagon; CM: Morse Taper; TI: Internal Triangle

**WARNING:** All Riellens Healing and Protective Covers have fittings dimensionally compatible with HE, HI, CM and TI connection implants.

#### 3. Specifications and technical characteristics

Riellens Healing and Protective Covers are made of titanium in compliance with ASTM F136 and ISO 5832-3 (Standard Specification for Wrought Titanium-6Aluminum-4Vanadium) standards, have a cylindrical shape and a hexagon socket (1.2mm) or Unigrip on the upper end and are supplied not-sterile.

#### 4. Method of Presentation

##### Package

Riellens Healing and Protection Covers are individually packaged in blister packaging, ensuring the integrity and cleanliness of the product until the use by the professional. Information such as description, dimension, date of manufacture and batch is available on the product label.

**CAUTION** -Always use the handle spot of the package, avoiding falls and direct contact with contaminated places.

#### 5. Handling

The Healing Cap and Protective Cover must be used for a short period of time and its removal is recommended within 30 days after implementation.

##### Healing Cap

Place the Healing Cap on the implant and apply the recommended torque (specific for each prosthetic interface).

#### - Protective Cover

Place the protective cover over the mini abutment or conical abutment and apply the recommended torque (specific for each prosthetic interface). The protective cover is used for a short period of time and its removal is recommended within 30 days after implementation.

#### 6. Recommended Torques / Used Wrenches

Table 2 - Correspondence Table Healing/Protective Cover x Wrench

COMPONENTS	TYPES OF THREADS	PLATFORM		RECOMMENDED TORQUE	MAXIMUM ALLOWABLE TORQUE	USED WRENCH	
						HEX. 1.2	UNIG (Torx)
	Thread M1.4	HE	MU(RP)	20 N.cm	26 N.cm	X	X
	Thread M1.6	HE HI CM	HEB (NP) HI CE (NP, RP, WP, 6.0) BL (NC, RC); NA (NP)	25 N.cm	32.5 N.cm	X	X
	Thread M1.8	HE HI CM	MU(WP) MS (Ø 3.75) TI RS (NP) CMN (Ø 3.3, 4.5) CMD (Ø 3.8)	25 N.cm	32.5 N.cm	X	X
	Thread M2.0	HE TI CM	3HE (NP; WP), HEB (RP) RS (RP, WP, 6.0) TL, ST, SI (RN, WN) NA (RP) CMD (4.5/5.0)	32 N.cm	41.6 N.cm	X	X
	Thread M2.5	HE TI	HEB (WP) RS (RP, WP)	32 N.cm	41.6 N.cm	X	X

#### 7. Contraindications:

The use of Riellens Healing and Protective Covers is contraindicated for patients who are allergic to any chemical element that comprise their raw material, such as: V, Ti, Al, O, N, Fe, H and C.

#### 8. Cleaning of the Components:

Riellens provides all its Healing and Protective Caps sanitized. Any component used that must be returned to the after-sales service shall be shipped sterile after pre-disinfection, cleaning and decontamination of the office.

#### 9. Sterilization

Healing and Protective Covers are non-sterile, so they must be sterilized in an autoclave according to professional protocol and the manufacturer's instructions before being used on the patient.

Riellens recommends placing the components in their own packages for sterilization in an autoclave. As the table below:

Method	Conditions
Autoclave, Humid heat	134° for 18min

#### 10. Precautions and Warning

Failure to follow the procedures outlined in this instruction for use may lead to one or all of the following complications: implant damage; components loosening; or poor healing of gingival tissue.

"REPROCESSING IS FORBIDDEN" the Healing and Protective Covers, the reuse of components will result in damage to the patient's health, who may come in contact with a contaminated product. The use of components or instruments that are not compatible will render all warranties void.

The correct use and handling of this product is the sole responsibility of the user. Riellens takes no responsibility for damages resulting from the incorrect use of its products.

The environment for the implementation of the prosthesis must be clean and suitable for the handling and use of health products.

If technical complaints or adverse events occur with Riellens products, please notify us through SAC.

**ATTENTION**  
Dental product intended for use by qualified professionals.  
Supplied NON-STERILE and must be sterilized before use.

#### 10. Warnings

Torsion superior to the recommended torque may result in damage to the Healing and Protective Covers fitting and fracture, causing prosthesis failure.

Torsion inferior to the recommended torque may result in component loosening, which may lead to poor healing of the gingival tissue.

#### 11. Caution

To tighten the Healing and Protective Caps, use the fixing wrench and finish with the torquemeter until the desired torque is obtained. Follow the instructions from the manufacturer of the equipment.

Ensure that the Healing and Protective Covers are correctly aligned to prevent wear on the threads.

#### 12. Adverse Effects

Possible complications:  
- Pain, swelling, phonetic difficulties, gingival inflammation, localized or systemic infections or aesthetic problems.

#### 13. Pos-implementation Care

Advise the patient to see the dental surgeon periodically.

#### 14. Condition of Storage

Always store at room temperature between 15°C and 35°C in the original package. Do not expose to direct sunlight.

#### 15. Disposal of the Product

For disposal of Healing and Protective Covers used or considered inadequate, it is necessary to follow the current legal procedures for disposal of products, and they must be decharacterized, whether by being cut, kneaded or filed

#### 16. Expiration Date and Batch

See the manufacture date, expiration and batch on the package.

#### 17. INSTRUCTION FOR USE WARNING

This INSTRUCTION FOR USE is available in non-printed (electronic) format, through the electronic address of the manufacturer [www.riellens.com.br](http://www.riellens.com.br)

The INSTRUCTIONS FOR USE are indexed on the website through the respective product TRADE NAME, informed on the label of the product purchased. If it is in the professional's interest, the INSTRUCTIONS FOR USE may be provided in printed form at no additional cost. The request must be made to the manufacturer's Customer Service (SAC).

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